TORQUE WRENCH

Operation & Maintenance Instructions

You will receive maximum value from your Torque Wrench if you acquaint yourself with these instructions for use and maintenance guidelines. Please read and follow these instructions carefully.

DESCRIPTION

The Dental Implant Driver Torque Wrench is a non-adjustable, manual, 4.0mm square, box-type torque wrench. Each wrench features a torque limit that is pre-set to apply a specific amount of torque when seating dental implant screws.

CLEANING & STERILIZATION

- Before initial use and after each procedure, clean the Torque Wrench with a wet towel or disinfectant wipe. Rinse under warm tap water and wipe dry. Before sterilization, spray an approved instrument lubricant into the opening at the break of the wrench.
- The Torque Wrench may be sterilized by steam autoclave. Sterilize for 20 minutes at 135° C (275° F), with the handle in the “broken” position. After sterilization, re-set the torque break in the handle, then manually “break” the torque again to ensure that the wrench is operating correctly.

OPERATION

- The Torque Wrench is designed to tighten or insert in a clock-wise direction, and loosen or unscrew in a counter-clockwise direction. When the wrench is rotated in the direction of the black arrow on the handle (clock-wise), the wrench’s torque limit will break at a pre-set level, and the head of the wrench will break away from the handle. The level of maximum torque is set into the wrench during manufacturing, and can only be adjusted during professional calibration. The level of torque is indicated on the handle, and expressed numerically in Newton Centimeters (Ncm).
- When the wrench is rotated against the direction of the black arrow on the handle (counter-clockwise), the wrench head will not break away from the handle. Use this direction for unscrewing or loosening only.
- The Torque Wrench is designed to work with all 4.0mm, square head dental implant drivers. Please consult the instructions included with the implant system for instructions on setting screws and inserting drivers.

CALIBRATION

The Torque Wrench should be calibrated every twelve months or after 100 auto-clave cycles to ensure accurate torque control. Please consult your authorized distributor for additional instructions.

WARRANTY

The Torque Wrench is warranted to be free from defects in material and workmanship from a period of one (1) year from date of purchase. This warranty only applies if the product has been operated in accordance with these instructions and the instructions of the implant system manufacturer. The warranty will not apply if the product has been subject to abuse or misuse, nor does it apply to normal wear. Sole liability under this warranty is limited

to repair or replacement, at the option of the manufacturer. No other warranties are expressed or implied. Any defective product must be returned to the factory (freight pre-paid) with proof of purchase enclosed.

DESCRIPTION	PRE-SET TORQUE SPECIFICATION
Torque Wrench 15 Ncm	15 Ncm ± .75 Ncm
Torque Wrench 20 Ncm	20 Ncm ± 1.0 Ncm
Torque Wrench 25 Ncm	25 Ncm ± 1.3 Ncm
Torque Wrench 30 Ncm	30 Ncm ± 1.5 Ncm
Torque Wrench 35 Ncm	35 Ncm ± 1.8 Ncm

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INFORMATION POUR RE-COMMANDER

DESCRIPTION	Spe’cification de torsion pre’-e’tablie
Torque Wrench 15 Ncm	15 Ncm ± .75 Ncm
Torque Wrench 20 Ncm	20 Ncm ± 1.0 Ncm
Torque Wrench 25 Ncm	25 Ncm ± 1.3 Ncm
Torque Wrench 30 Ncm	30 Ncm ± 1.5 Ncm
Torque Wrench 35 Ncm	35 Ncm ± 1.8 Ncm

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